

1



Take out the product.

2



Puncture with the blade of the inner needle facing upwards.

Note: Be sure to hold the inner needle hub and take care that the inner needle does not move backward when puncturing.

3



Carefully check and verify the entry of the needle tip into the blood vessel by checking for blood flashback in the needle hub.

Note: Replace the needle with a new product when attempting re-puncture due to access failure.

Advance the catheter tube forward into the blood vessel while stabilizing the inner needle with the catheter.


Withdraw the inner needle.

4




Clamp the clamping tube tightly with non-serrated forceps.

5

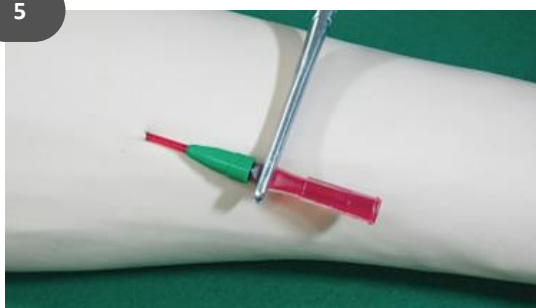


Remove the rubber adapter.

6

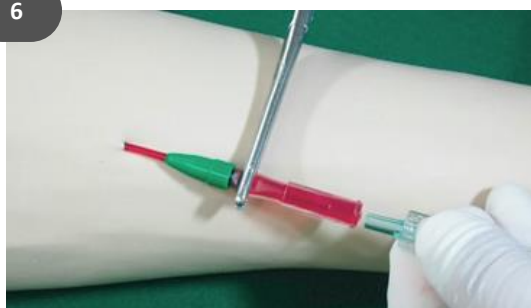

Release the air remaining in the catheter by adjusting the clamping tube.

7

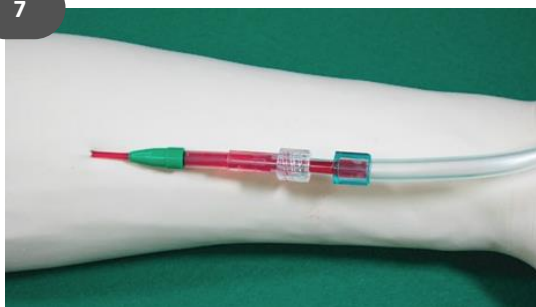

Firmly connect the needle connector to the blood circuit and remove the clamp. Stabilize the position of the catheter and the tube with adhesive tape.



**MEDIKIT**

MEDIKIT EUROPE GMBH

Gerbermühlstraße 7

60594 Frankfurt am Main

[www.medikit.co.jp/english/](http://www.medikit.co.jp/english/)

MANUFACTURER: Togo Medikit Co. Ltd.